

DECENT SIMULATORS

AIRWAY MANAGEMENT

TASK TRAINER | C0002

| | | | |
|------------------------|---|---|---|
| WHAT'S INCLUDED | CARRY BAG, USB, NECK SKIN (X2), LUBRICANT, CRIC MEMBRANE (X2), TEETH - DIFFERENT PATTERNS (X4), AIRWAY BLOCKAGES | |  |
| COMPANY DETAILS | UNIT 13D RATHENRAW INDUSTRIAL ESTATE ANTRIM, BT412SJ, UNITED KINGDOM W: WWW.DECENTSIMULATORS.COM |  | |

PREPARATION

PREPARATION AND ASSEMBLY

1. LUBRICATE THE AIRWAY ENTRY USING THE PROVIDED LUBRICANT.
2. INSERT THE TRACHEA (WITH LUNGS AND STOMACH) INTO THE AIRWAY.
3. SECURE THE ASSEMBLY BY HOOKING THE SIDE ELEMENTS ONTO THE TRACHEA'S SIDE PINS.

ADJUSTING TRAINING DIFFICULTY

1. TEETH: SWAP THE TOP TEETH TO INCREASE INTUBATION DIFFICULTY.
2. NECK STIFFNESS: USE THE NECK SCREW TO STIFFEN THE NECK—BE CAREFUL NOT TO OVERTIGHTEN.
3. AIRWAY POSITION: ADJUST THE AIRWAY'S POSITION BY TURNING THE SPINE SCREW TO MAKE IT MORE ANTERIOR.

USAGE

1. BEFORE USE:

LUBRICATE THE AIRWAY WITH THE PROVIDED LUBRICANT. THIS IS KEY FOR PROPER FUNCTION AND TO PREVENT DAMAGE.

2. ALTERNATIVE:

IF YOU RUN OUT OF THE PROVIDED LUBRICANT, YOU CAN MIX HAND SOAP OR DISHWASHING LIQUID WITH WATER AS AN ALTERNATIVE.

STORAGE

1. BEFORE STORAGE:

REMOVE THE LUNGS FROM THE AIRWAY.

2. CLEANING:

WASH ALL PARTS IN THE SINK USING SOAP AND WATER.

3. DRYING:

MAKE SURE EVERY PART IS COMPLETELY DRY BEFORE STORING TO PREVENT MOLD GROWTH.

REPAIR INSTRUCTIONS

- PLASTIC PARTS: REPAIR USING ANY CYANOACRYLATE GLUE (E.G., 'SUPERGLUE').
- SILICONE PARTS: REPAIR USING SILICONE GLUE (E.G., 'SIL-POXY'). FOR ADDED STRENGTH, APPLY NYLON MESH OVER THE DAMAGED AREA.

REPAIR KITS ARE AVAILABLE AT WWW.DECENTSIMULATORS.COM/STORE OR THROUGH YOUR LOCAL DISTRIBUTOR.