


SALAD

MODULE | MP0003

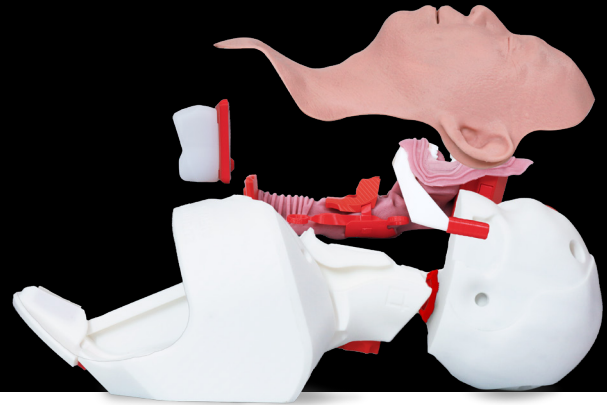

Add simulated blood and secretions to any V2 airway and practise suction-assisted laryngoscopy and airway decontamination (SALAD).

The module is quick to set up, easy to reset and simple to clean.


Use it with standard or difficult airways to make airway management more challenging.

KEY FEATURES



*SIMPLE SETUP

A compact, straightforward system that is quick to prepare for training.

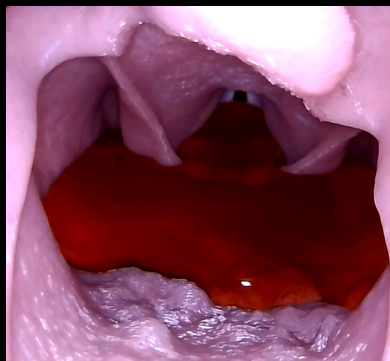

*WORKS WITH ALL V2 AIRWAYS

Use the module with any V2 airway, including difficult airway options. This allows you to add contamination to an existing difficult airway scenario.



*EASY TO CLEAN


Remove the module, disconnect the tubing and rinse the parts with water after training.

SET UP THE MODULE



FIT THE MODULE

Swap the lung module for the SALAD module.

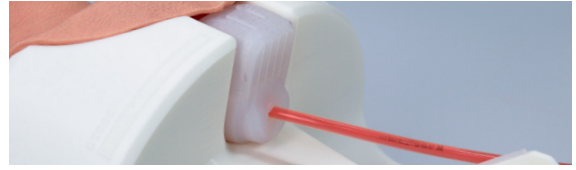

ADD FLUID

Inject fluid through the SALAD port.

SUPPORTS THE FOLLOWING LEARNING OBJECTIVES:

CONTAMINATION

Maintain a usable airway view during ongoing contamination.

SUCTION

Position and use suction effectively during laryngoscopy.

PROCEDURES

Coordinate suction, laryngoscopy and tracheal intubation.

KIT INCLUDES:

- * SALAD insert
- * Pressure sprayer
- * Tubing
- * Silicone mat
- * Absorbent cloth
- * Simulated blood powder
- * Measuring scoop
- * Instructions for use



WATCH
THE SALAD
MODULE IN
USE



CONTACT US VIA
[DECENTSIMULATORS.COM](https://www.decentsimulators.com)