

DECENT SIMULATORS

CRICOTHYROIDOTOMY

TASK TRAINER | C0003

WHAT'S INCLUDED	CARRY BAG, USB, FAT PADS (X4), FAT PADS BLOOD FILLED (X4), SPARE NECK SKINS (X2), CRIC INSERTS DIFF. PATTERNS (X4), DUCT TAPE (CRIC MEMBRANE), BORESCOPE CAMERA		
COMPANY DETAILS	UNIT 13D RATHENRAW INDUSTRIAL ESTATE ANTRIM, BT412SJ, UNITED KINGDOM W: WWW.DECENTSIMULATORS.COM		

PREPARATION

1. PREPARE THE CRIC INSERT:

WRAP THE PROVIDED DUCT TAPE AROUND IT SEVERAL TIMES UNTIL SECURE.

2. INCREASE THE CHALLENGE:

PLACE A FAT PAD OVER THE CRIC INSERT, TUCK ITS SIDES UNDER THE SILICONE “MUSCLE,” AND COVER IT WITH THE NECK SKIN. FOR AN EXTRA CHALLENGE, USE PADS FILLED WITH SIMULATED BLOOD.

3. VIEW FROM INSIDE THE LARYNX:

INSERT THE PROVIDED USB CAMERA INTO THE OPENING AT THE TOP OF THE TASK TRAINER. CONNECT IT TO YOUR MOBILE PHONE FOLLOWING THE INSTRUCTIONS ON THE PACKAGING, THEN GENTLY PUSH IT FORWARD UNTIL YOU CLEARLY SEE INSIDE THE LARYNX.

USAGE

1. AFTER A PROCEDURE:

REPLACE THE TAPE ON THE CRIC INSERT AND ADJUST THE NECK SKIN.

2. SETUP:

ENSURE THAT, ONCE THE NEW CRIC INSERT IS PLACED IN THE BASE, IT CONTACTS THE TRACHEA—THIS ALIGNMENT ALLOWS THE BOUGIE TO SLIDE SMOOTHLY THROUGH THE CONNECTION POINT.

STORAGE

1. CLEANING:

WASH ALL PARTS IN THE SINK USING SOAP AND WATER.

2. DRYING:

MAKE SURE EVERY PART IS COMPLETELY DRY BEFORE STORING TO PREVENT MOLD GROWTH.

REPAIR INSTRUCTIONS

- PLASTIC PARTS: REPAIR USING ANY CYANOACRYLATE GLUE (E.G., ‘SUPERGLUE’).
- SILICONE PARTS: REPAIR USING SILICONE GLUE (E.G., ‘SIL-POXY’). FOR ADDED STRENGTH, APPLY NYLON MESH OVER THE DAMAGED AREA.

REPAIR KITS ARE AVAILABLE AT WWW.DECENTSIMULATORS.COM/STORE OR THROUGH YOUR LOCAL DISTRIBUTOR.